

## H6/H7 HAND SCALER

128-007



### PRODUCT DESCRIPTION

---

## Double ended hand scaler with scythe shape to fit the tooth surface and a balanced grip handle

#### Key Features

- A sickle hand scaler - the scythe shape fits the curved tooth surface.
- A double ended instrument - each end being the mirror image with opposite complementary curvature - left and right.
- Hand scalers have a sharp tip and a more robust working end compared to a curette.
- A balanced grip handle which has a large diameter for comfortable grip and good control.

#### Uses

- Scaling teeth above the gum line ONLY, i.e. supragingivally, so removing calculus (tartar)

#### How to use

- The instrument is held in a modified pen grip
- The working end is placed against the tooth surface just above the gumline with the flat face at 90 degrees so that the sharp edge contacts. The scaler is pulled across the tooth, usually up the crown from the gumline towards tip.

### ADDITIONAL INFORMATION

**Animal**

Mixed

**SKU**

128-007

**Dental Instrument Type**

Other

### Good to know...

At Burtons we know you have a choice to shop with us, that is why we always aim to delight our customers with a fast, easy, shopping experience. Making and sourcing only the best products we can find to fit your needs. Price Matching: We regularly check all our prices against competitors so you don't have to. If you do find a like for like product cheaper we will aim to not just match it, but beat it! Backorders: All back ordered items will be ordered and shipped as quickly as we can to you. Special order items may take up to 4 weeks. Once ordered a member of our team will be in contact to keep you informed of estimated delivery dates. Returns Policy: You may return new, unworn or unused products within fourteen (14) days of delivery for a full refund of the cost of the goods, or an exchange if preferred. This is on top of your statutory rights