

## PERIOSTEAL ELEVATOR GOLDMAN FOX PATTERN

128-008



## PRODUCT DESCRIPTION

---

### Double ended periosteal with 2 different sized blades

#### Features

- Double ended with small and larger “blades” (A&B)
- Deluxe (large diameter) handle for better grip

#### Uses

- Raising soft tissue flaps (mucoperiosteal) by separating the periosteum (with all soft tissues above) from the bone.
- As a gingival retractor and guard e.g. while extracting teeth to avoid soft tissue trauma by the dental drill or elevators.

#### How to use

- Use with the flat side of the blade towards the bone surface and the rounded back of the blade under the soft tissue.
- Do not use a flat angle to the bone surface as this is more likely to split the soft tissue planes
- Hold at an obtuse angle
- Use a twisting action similar to peeling an orange with a fingernail

## ADDITIONAL INFORMATION

---

**Animal**

Mixed

**SKU**

128-008

### Good to know...

At Burtons we know you have a choice to shop with us, that is why we always aim to delight our customers with a fast, easy, shopping experience. Making and sourcing only the best products we can find to fit your needs. Price Matching: We regularly check all our prices against competitors so you don't have to. If you do find a like for like product cheaper we will aim to not just match it, but beat it! Backorders: All back ordered items will be ordered and shipped as quickly as we can to you. Special order items may take up to 4 weeks. Once ordered a member of our team will be in contact to keep you informed of estimated delivery dates. Returns Policy: You may return new, unworn or unused products within fourteen (14) days of delivery for a full refund of the cost of the goods, or an exchange if preferred. This is on top of your statutory rights