

## **COCOON FORCED AIR WARMING BLANKETS**

#### CWS-BLANKETS-BUNDLE\*







# PRODUCT DESCRIPTION

# Made from soft, comfortable, non-woven fabric, the Cocoon Blanket's unique perforations provide uniform air distribution effectively.

Universal inlet port ties on each blanket ensure that the hose remains firmly in place and the strong fabric allows the use of the same blanket on the patient, from pre-warming, through surgery, to recovery. The blankets are designed for single use and can be quickly disposed of after surgery. They are also easy to store and transport, making them the ideal solution for any environment. Some blankets have multiple inlet port locations for the best access. (See below for details on each blanket).

Combined with our Cocoon Forced Air Warming System, these blankets provide quick, effective patient warming.

### Key Features of Cocoon warming blankets.

- Comfortable and lightweight fabric
- Microscopic perforations for quick warming without excess air being blown around the surgical site.
- Universal inlet port can be used with other blankets.
- Although these are single-use blankets, the fabric is strong enough to use multiple times throughout the patient's treatment, from surgery to recovery.
- Universal ties help to secure the blankets to the blower.

#### Usage Guide:

- An incontinence pad may be placed under the animal in the central area.
- Place the blanket either under or over depending on the blanket model.
- White side of the blanket faces the animal.
- Insert the heater hose into any blanket inlet port.
- Don't occlude the inflation tube at the blanket inlet or when closing a cage door.
- Monitor animal temperature frequently.
- Replace blanket if wet or soiled.

https://burtonsveterinary.com/



• Position the animal either dorsal or lateral.

#### Blanket Models - all blankets come in boxes of 10.

- VET001 Under Blanket -M/L 126cm L x 79cm W 3 hose ports Usage: Anaesthesia / Surgery
- VET002 Under Blanket S 86cm L x 79cm W 3 hose ports Usage: Anaesthesia/ Surgery
- VET003 Over Blanket M/L 123cm L x 83cm W 2 hose ports Usage: Cage/recovery/ critical care
- VET004 Over Blanket S 66cm L x 86cm W 2 hose ports Usage: Cage/ anaesthesia/ critical care/ recovery
- VET005 Over Blanket L 100cm L x 120cm W 1 hose port Usage: Anaesthesia/ recovery/ critical care.
- VET006 Over Blanket XL 100cm L x 195cm W 1 hose port Usage: Anaesthesia/ recovery/ critical care
- VET007 Tube Blanket M/L 180cm L x 14cm W 1 hose port Usage: Cage/ anaesthesia/ recovery/ critical care
- VET008 Tube Blanket S/M 99cm L x 14cm W 1 hose port Usage: Cage, anaesthesia, recovery/ critical care
- **VET009** Over Blanket L with extra-long inlet port 100cm L x 12cm W 1 hose port Usage: anaesthesia/ recovery/ critical care ideal for post-surgery with an extra-long inlet port for easy access to the patient inside a kennel.

## ADDITIONAL INFORMATION

SKU	CWS-BLANKETS-BUNDLE*
Animal	Mixed

#### Good to know...

At Burtons we know you have a choice to shop with us, that is why we always aim to delight our customers with a fast, easy, shopping experience. Making and sourcing only the best products we can find to fit your needs. Price Matching: We regularly check all our prices against competitors so you don't have to. If you do find a like for like product cheaper we will aim to not just match it, but beat it! Backorders: All back ordered items will be ordered and shipped as quickly as we can to you. Special order items may take up to 4 weeks. Once ordered a member of our team will be in contact to keep you informed of estimated delivery dates. Returns Policy: You may return new, unworn or unused products within fourteen (14) days of delivery for a full refund of the cost of the goods, or an exchange if preferred. This is on top of your statutory rights

https://burtonsveterinary.com/ Page 2 Sales: 01622 834300 | Service: 01622 834350