

VETRONIC SAV04 SMALL ANIMAL VENTILATOR - CLEARANCE

VSL-SAV04-CL-2





PRODUCT DESCRIPTION

Safe, simple, pressure cycled ventilation for small domestic animals up to 12kg in weight

The SAV04 incorporates a host of features for improved control of IPPV

The SAV04 is an enhanced version of the long-standing SAV03 ventilator and uses the same principles of simple, pressure-cycled ventilation.

In the redevelopment of the ventilator, features have been added to improve patient control and to improve ventilation status feedback. For example, there is now an automatic over-pressure alarm that will sound if the target pressure is exceeded by 8cm. There is also an alarm if the patient becomes disconnected from the ventilator, or if no back-pressure is detected during the inspiratory phase.

These alarms are designed to be informative and are accompanied by explanatory text on the main screen. In addition, an audible beep is produced on every successful breath. Both this beep and the alarm beeps can be individually silenced if required. There is a new valve design to accompany the new ventilator and this has two major benefits.

Firstly, the valve can be automatically zeroed on any other SAV04 machine allowing valves to be swapped or replaced easily.

Secondly, a larger orifice in the exit port of the valve means the port can now support larger animals. The 12mm diameter exit orifice will now allow unrestricted expiration in animals up to 40kg.

It must be remembered the principle of operation is still based around a T-piece, so proportionately higher fresh gas flow rates will be required. However, with the advent of oxygen generators, running such a system on 8-10L is feasible for short-term or emergency use. The new display now provides more information to the user. A central rising bar graph shows the pressure in the patient's airway, enabling quick assessments to be made on system and/or ET tube leaks. Each inspiratory period is now measured and displayed on the screen as is the measured respiratory rate.

TheSAV04 provides a simple, yet safe, way to ventilate your smaller, most vulnerable patients

Watch our video giving an Introduction to the SAV04 Ventilator here

https://burtonsveterinary.com/ Page 1 Sales: 01622 834300 | Service: 01622 834350



Key features:

- Incorporating alerts and alarms for ventilation status
 - Patient disconnect
 - Over-pressure
 - Apnoea
 - · Valve disconnect or failure
- Measurement of inspiratory time and respiratory rate
- Uses the same VitalMonitor Software as all Vetronic monitors, making it easy to use them together for live data viewing and patient records
- Microprocessor controlled
- Select-and-commit entry to prevent false changes
- Easy and intuitive to use
- Compact, rugged unit
- Internal battery for portable/mobile use
- Clear, bright visual display
- Designed and manufactured in the UK

S/N: V-22-1-23-490

ADDITIONAL INFORMATION

Animal	Small Animal
SKU	VSL-SAV04-cl-2

Good to know...

At Burtons we know you have a choice to shop with us, that is why we always aim to delight our customers with a fast, easy, shopping experience. Making and sourcing only the best products we can find to fit your needs. Price Matching: We regularly check all our prices against competitors so you don't have to. If you do find a like for like product cheaper we will aim to not just match it, but beat it! Backorders: All back ordered items will be ordered and shipped as quickly as we can to you. Special order items may take up to 4 weeks. Once ordered a member of our team will be in contact to keep you informed of estimated delivery dates. Returns Policy: You may return new, unworn or unused products within fourteen (14) days of delivery for a full refund of the cost of the goods, or an exchange if preferred. This is on top of your statutory rights

https://burtonsveterinary.com/ Page 2 Sales: 01622 834300 | Service: 01622 834350